

USA Sport & Health®

A DIVISION OF CAPITAL PROSTHETIC AND ORTHOTIC CENTER INC.

Introduces the LUMB-AIR® COMPONENTS FOR THE MULTI BRACE

MODULAR
UNIVERSAL
LUMBAR
THORACIC
INNOVATION

LSO

TLSO

TLSO
STERNAL
SECTION

THE LUMB-AIR® COMPONENTS

THE
INFLATABLE
AIR-BAG

The
LUMB-AIR®
MULTI BRACE

**Controlled
pressure for
pain relief
and spinal
correction**

COLLAPSED


INFLATED



US patent number 6,932,780, US patent number 8,920,352 and others pending

APPLICATION of the LSO brace

1.

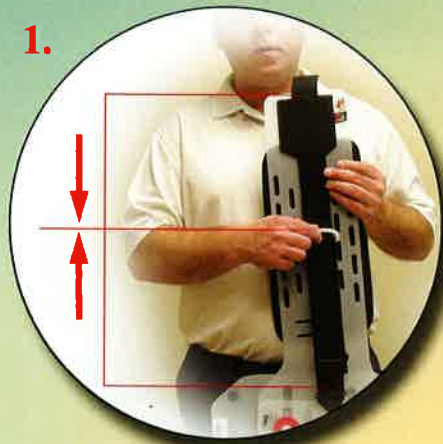

Place the buckles of the closure straps in the middle of each side panel.

2.



Fully extend both lateral panels to open the LSO brace.

3.



Position the LSO on your lower back with the bottom of the lumbar panel at the top of the coccyx.

4.



Wrap the LSO around the abdomen securing the lateral panels together.

5.



Grab the closure straps pulling away from the body at a 45° angle.

6.




Bring the closure straps to the front of the brace and attach. Straps may overlap.



HIGH TECH CARE WITH A TENDER TOUCH®

US patent number 6,932,780, US patent number 8,920,352 and others pending

APPLICATION of the TLSO brace

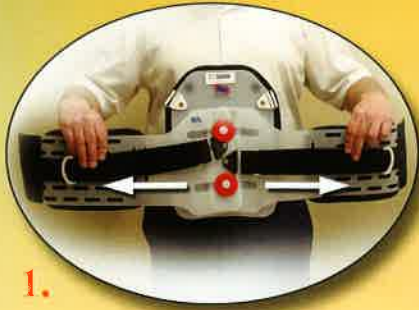

1.

Place the buckles of closure straps in middle of each side panel. Then fully extend each side panel.



2.

Position the sternal section against your body, then attach the side panels snugly with the hook and loop.



3.

Grab closure straps and pull away from your body at a 45° angle.



4.

Bring closure straps to the front of the brace and attach. Straps may overlap.



5.

Attach the thoracic straps comfortably *under the arm*.



6.

Properly fitted brace

REMOVING the TLSO brace



1.

Remove one thoracic strap



2.

Loosen closure straps



3.

Peel away one side panel from the front piece. Leave the other side attached.



4.

Fully extend side panels. Brace is ready to wear again.

Why use the USA SPORT AND HEALTH MULTIBRACE?

This is a bracing system that meets physician and government requirements

The lightweight plastic bracing, combined with custom-fit technology offers the best alternative to meet your patients' needs

It provides maximum comfort for maximum support


One bracing system for most spinal pathologies

- *Lightweight and easy to apply*
- *Can be worn under or over clothing*
- *Low profile*
- *Ventilated (prevents heat build-up)*
- *So unique that it has its own patent.*

CLEANING INSTRUCTIONS

Remove the pads, hand wash in luke warm water and air dry.

PLACEMENT OF THE PADS

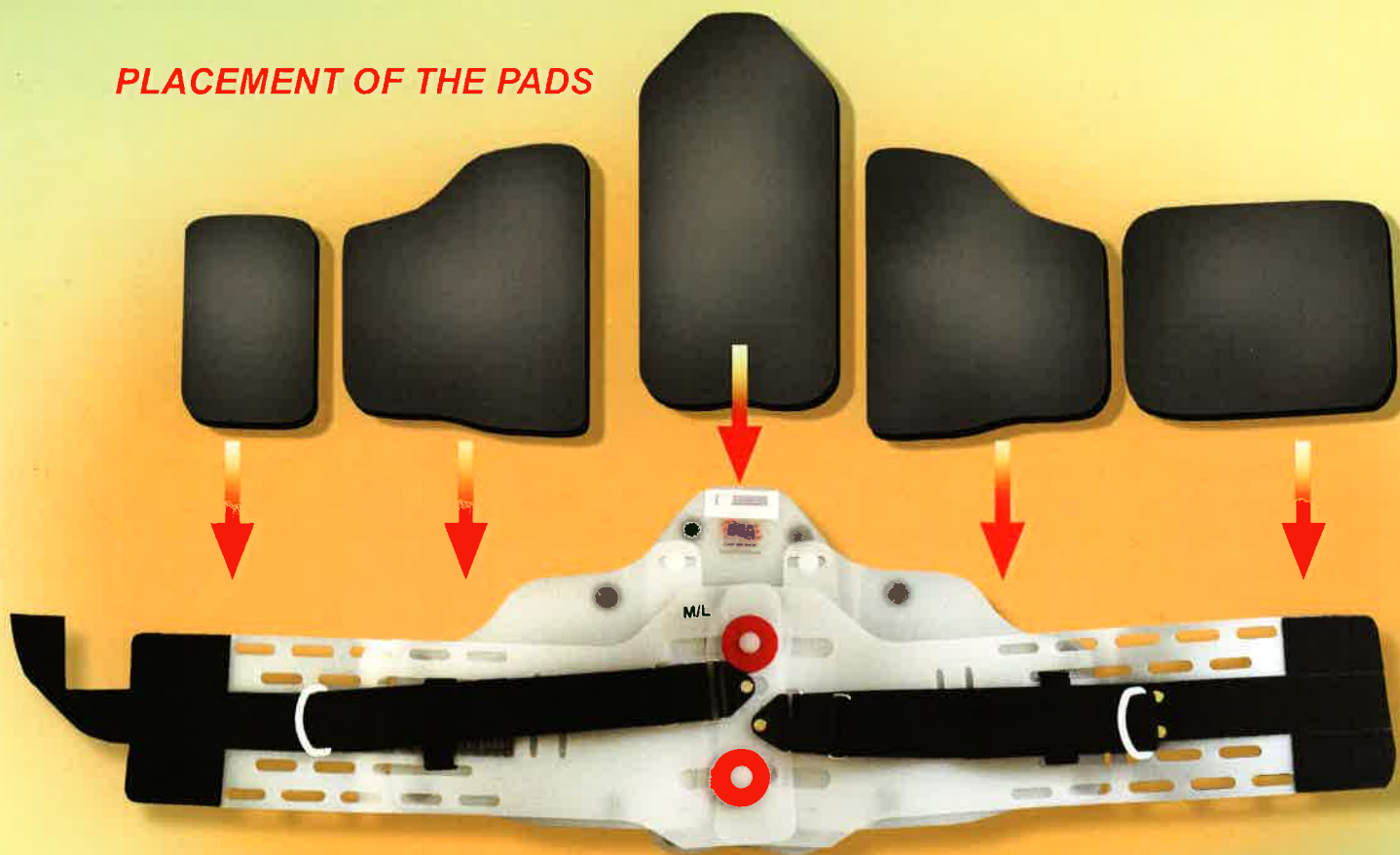

SIZING CHART

SMALL	32-36"	81-91 cm	WAIST
MEDIUM	36-40"	91-102 cm	WAIST
LARGE	41-45"	104-114 cm	WAIST
X-LARGE	46-53"	117-135 cm	WAIST
2X	54-62"	137-157cm	WAIST

CAPITAL PROSTHETIC AND ORTHOTIC CENTER, INC.

4678 Larwell Drive
Columbus, OH 43220

614-451-0446

(800) 215-0764

dkozersky@capitalprosthetics.net

**HIGH TECH CARE WITH A
TENDER TOUCH®**

US patent number 6,932,780, US patent number 8,920,352 and others pending

