

USA Sport & Health

A DIVISION OF CAPITAL PROSTHETIC AND ORTHOTIC CENTER INC.

Introduces The LSB™ Lumbar Sacral Belt

WITH LUMB-AIR® ADJUSTMENT

INDICATIONS:

- Acute and chronic low back pain
- Discogenic pain
- Muscular strain
- Post surgical
- Patients who need lumbar support for work or play



BRINGING ORTHOPEDIC
TECHNOLOGY
TO EVERYDAY LIVING



Sports

Industry


Daily Living

Office



THE LUMB-AIR® COMPONENTS

The
LUMB-AIR®
MULTI BRACE

Controlled
pressure for
pain relief
and spinal
correction

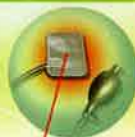

THE
INFLATABLE
AIR-BAG

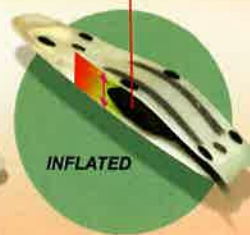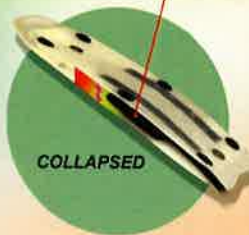

US patent number 6,932,780,
US patent number 8,920,352
and others pending

Application of the LSB™

1.



Place the buckles of the closure straps in the middle of each side panel.

2.



Fully extend the LSB™.

3.



Wrap the LSB™ around the waist and attach the front Velcro®. Make sure that the size indicator is facing upward.

4.



Grab the tension straps and pull forward at 45° angles.

5.



Attach the Velcro® at the sides.

SIZING CHART

SMALL	32-36"	81-91 cm	WAIST
MEDIUM	36-40"	91-102 cm	WAIST
LARGE	41-45"	104-114 cm	WAIST
X-LARGE	46-53"	117-135 cm	WAIST
2X	54-62"	137-157cm	WAIST

4678 Larwell Dr.
Columbus, OH 43220
800-355-6492
www.capitalprosthetics.net



HIGH TECH CARE WITH A TENDER TOUCH®